




NART ELEVATED LIFT USER GUIDE

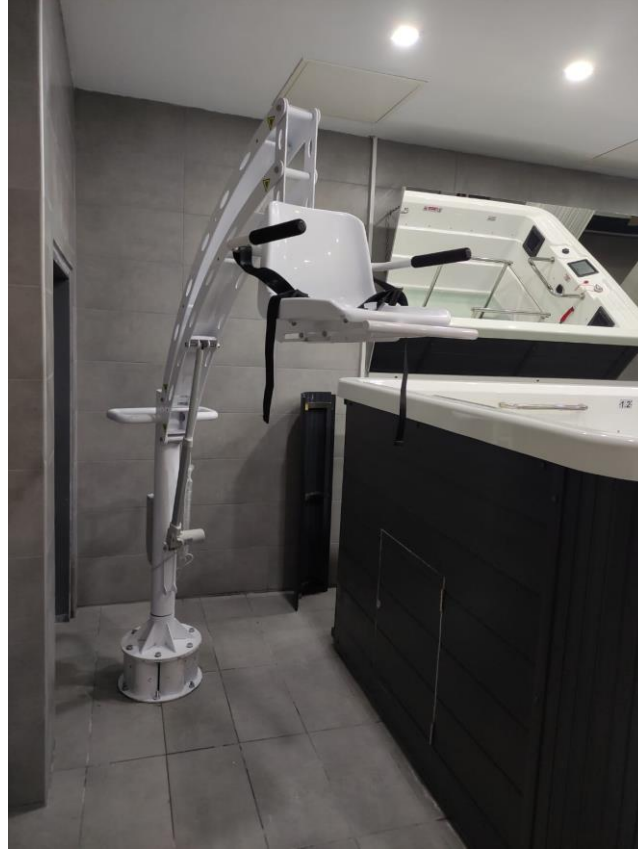

NART Access Disabled and Elderly Elevators Company Factory: Muradiye Mh. 10 (OSB) Sk. C6 Blok
No:9/6A Yunusemre / MANİSA
T: +90 0236 231 7711

Mail: info@nartaccess.com

www.nartaccess.com

RULES TO BE COMPLIED AND INFORMATION ON USE ERRORS

- * Read and follow all instructions
- * To use the elevator safely, you must follow the installation and operating instructions
- * Do not allow children to play with and around the elevator
- * Do not expose the electronic components to direct water pressure.
- * For safety reasons, do not use in a situation where no one can help you in an emergency
- * Do not use without safety belt
- * Press the emergency button in an emergent situation

1.GENERAL FEATURES

1.a)Usage Area of the Machine:


The NART FREE lift is a pool lift for disabled and elderly people which is made of 304 quality stainless steel with electrostatic powder paint on it. The NART FREE LIFT has rechargeable battery and actuator. It has 360 degree seat rotation. The NART FREE LIFT can be used on all pools.

1.b) Machine Specifications:

- 150 kg carrying capacity
- All stainless steel construction
- Water erosion, High temperature etc. Resistant to corrosion
- Wear and corrosion resistant electrostatic powder paint oven paint is used.
- 360 Degree seat rotation.
- Safety belt and adjustable footrest
- Waterproof hand control (IP 66 standard)
- Has actuator drive mechanism and battery power supply
- CE certificate is available
- ISO 9001 and ISO 13485 certificate is available
- Product is guaranteed for 2 years

1.c) Machine Components

- 1- Seat Belt
- 2- Control Unit
- 3- Battery
- 4- Battery charger
- 5- Control command

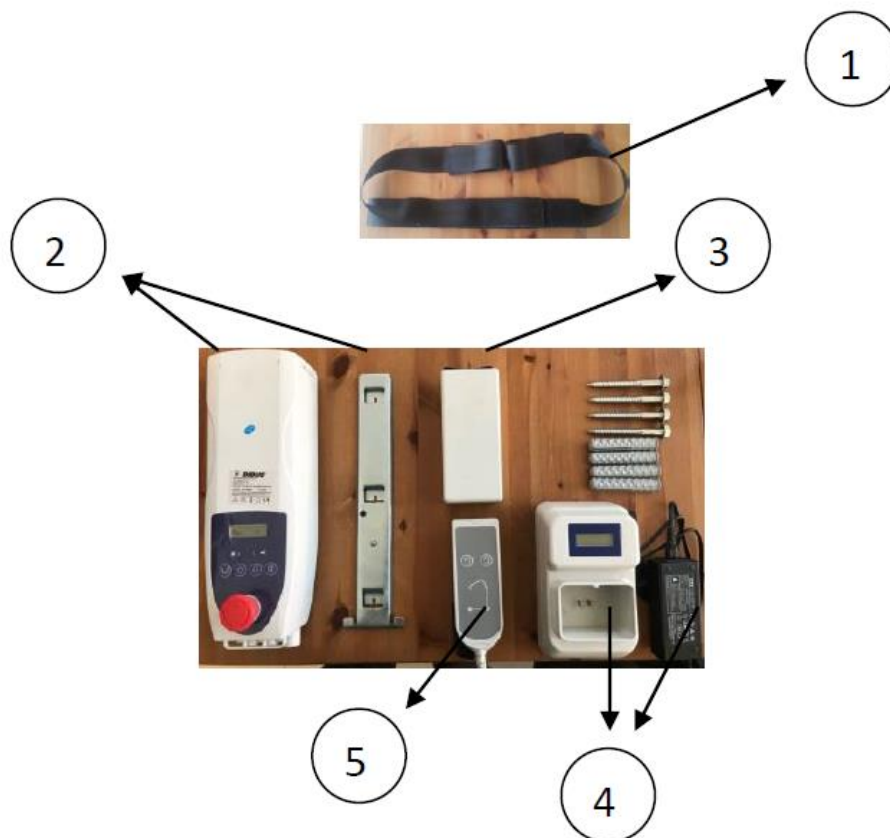

1.d) Purpose of the Manual:

The User Manual is designed to ensure the correct and safe use of the product and is intended for the safe and complete access of the pool to persons with disabilities when assembling and using the instructions

Please follow all rules.

2. MOUNTING AREA FEATURES AND STANDARDS

The working principle of the elevated pool lift is as follows:


✓To overcome a pool elevation above the ground, the user sits on the lift, rises from the pool level to the top, turns towards the pool with the help of a companion and descends towards the pool. 360 Degree rotation feature provides convenience.

✓The NART ELEVATED LIFT can be used in outdoor or indoor pools.

✓When deciding on the location of the NART ELEVATED LIFT fixing body, the fixing base should be installed a maximum of 100 cm away from the pool. This distance may be shorter depending on the height of the pool.

✓The user must decide the location of the elevator according to the height of the pool.

✓Please pay attention to the distance of the footrest in the pools those are not deeper than 140 cm.

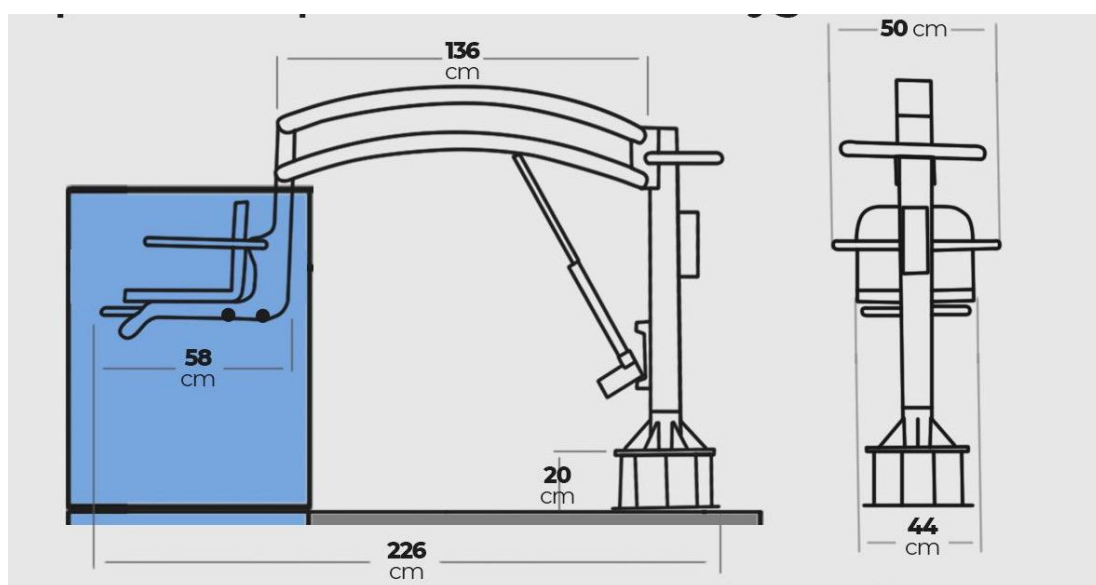

3. PERIODIC MAINTENANCE SCHEDULE OF THE POOL LIFT

CARE INSTRUCTIONS	DAILY	WEEKLY	MONTHLY
Clean with clean water after use.	X		
Charge the battery	X		
To check that it runs smoothly, run the elevator 1 time to complete its full movement without anyone on it.	X		
Check the lift for damage, corrosion and loose or missing material	X		
Check all connection points and check for damage or corrosion		X	
Check warning labels and replace if worn			X

Notes on Maintenance:

- * Clean the lift with clean water only, do not use pool water
- * Wipe with a clean, soft and non-abrasive cloth
- * Do not expose the elevator to electronic components or direct water pressure while cleaning the elevator.
- * Do not allow children to play with and around the elevator
- * Do not store the lift in a place where it can be impacted and can come into contact with chemicals

4. USE OF POOL LIFT

4.a) Entering the Pool

- Check the area where the elevator will operate and make sure there are no obstacles in the area.
- Sit in the seat and fasten your seat belt.
- Place your feet securely on the footrest.
- Press and hold the elevator up button on the remote control. (In accordance with the regulations, the elevator must stop as soon as the button is released.)
- When the desired height is reached, release the button and stop the elevator.

After reaching a height where you exceed the height of the pool, your companion must turn you in the direction of the pool.

- Press and hold the down button on the remote control to descend towards the pool.
- Unfasten your seat belt and enter the pool.
- Please press this red button in an emergency situation.



4.b) Exiting the Pool

- Check the area where the elevator will operate and make sure there are no obstacles in the area.
- Sit in the seat and fasten your seatbelt.
- Place your feet securely on the footrest.
- Press and hold the up button on the remote control.
- When the desired height is reached, release the button and stop the elevator.
- After reaching a height exceeding the height of the pool, your companion must turn you out of the pool with the steering body.
- To descend towards the ground, hold down the down button on the remote control and release it when you reach the ideal height.
- Unfasten your seat belt after the lift stops

7. SOLVING SIMPLE PROBLEMS

If the pool lift does not move ;

a) The control unit cable may be disconnected.

In this case, make sure that the cable of the device is connected to the control unit.

b) The connection elements between the control unit and the actuator may be dislodged

In this case, make sure that the connections between the control unit and the actuator are fully connected.

c) The battery (battery) may be discharged

The battery may be exhausted or out of use depending on its use.

In this case, try charging and re-operating the battery until it is fully charged with the charger.

As a result, if your problem is not solved, do not take any further action, please call our company for the necessary technical team guidance.

8. ISSUES TO BE CONSIDERED DURING TRANSPORTATION

For orders for which NART LIFT is responsible for transportation, the company takes responsibility for any situation that may arise due to transportation.

If a problem occurs due to transportation in the order of the buyer who organizes the transportation himself/herself, the buyer is responsible.

The user's responsibilities in the case of transport and transportation;

- Make sure that the product comes in the package (carton, pallet, etc.)
- Missing parts or faulty products must be reported to us within 5 business days.